

TRUTEST™ MULTI-PANEL DRUG TEST CUP

CLIA
WAIVED

Affordable, practical, accurate drug test cup fully integrated approach for on-site drug testing.



- 14 Available CLIA Waived Drugs
- Screw Top Lid with "Click" for Secure Closure
- Rapid Result Easy to Read One Drug Per Strip
- Up to 6 Specimen Validity Tests + Temp Strip
- 24 Month Shelf Life

RESULTS YOU CAN TRUST

POINT OF CARE MEDICAL
DIAGNOSTIC TESTING PRODUCTS

NOBLE
.....
MEDICAL INC

TRUATEST™

MULTI-PANEL DRUG TEST CUP

CLIA
WAIVED

STEP 1




STEP 2



STEP 3



2 Minutes

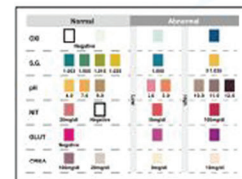

STEP 4



STEP 5



5 Minutes

Read Results
at 5 Minutes



NEGATIVE



POSITIVE



INVALID

TEST SPECIFICATIONS

TEST	CALIBRATOR	CUT-OFF (NG/ML)
Amphetamine (AMP)	D-Amphetamine	500
Amphetamine (AMP)	D-Amphetamine	1,000
Barbiturates (BAR)	Secobarbital	300
Benzodiazepines (BZO)	Oxazepam	300
Buprenorphine (BUP)	Buprenorphine	10
Cocaine(COC)	Benzoylcegonine	150
Cocaine(COC)	Benzoylcegonine	300
EDDP (Methadone Metabolites)	2-Ethylidene-1,5-dimethyl-3,3-diphenylpyrrolidine (EDDP)	300
Marijuana (THC)	11-nor-9-THC-9 COOH	50
MDMA(Ecstasy)	D,L-Methylenedioxymethamphetamine	500
Methadone (MTD)	Methadone	300
Methamphetamine (MET, mAMP)	D-Methamphetamine	500
Methamphetamine (MET, mAMP)	D-Methamphetamine	1,000
Morphine (MOP)	Morphine	300
Opiates (OPI)	Opiates	2000
Oxycodone (OXY)	Oxycodone	100
Phencyclidine (PCP)	Phencyclidine	25
Propoxyphene	Propoxyphene	300
Tricyclic Antidepressants (TCA)	Nortriptyline	1,000

DIRECTIONS FOR USE

ALLOW THE TEST CUP TO COME TO ROOM TEMPERATURE [15-30°C (59-86°F)] PRIOR TO TEST.

1. Tear the foil bag open, remove test cup and disposable gloves provided for donor. Label the device with donor information.
2. Open test cup lid. Urinate directly into the test cup. Be sure to fill up the test cup with the urine specimen between minimum 30ml to maximum 110ml (marked on the cup.)
3. After urine specimen has been collected, close the lid securely and return cup to collection official.
4. Collection Official should use gloves provided. Peel off label to reveal test result. Read test result at 5 minutes.

DO NOT INTERPRET RESULTS
AFTER 10 MINUTES

877.836.5713 NOBLEMEDICAL.COM
19525 Janacek Court, Ste. 104 | Brookfield, WI 53045

NOBLE
MEDICAL INC