

SPLIT SPECIMEN CUP[®]

MULTI-PANEL CUP

TEST WITH CONFIDENCE.

Innovative drug test cup design eliminates urine handling and donor tampering, providing a unique & efficient approach for on-site drug testing.



- Split-Specimen Collection
- Screw Top Lid for Secure Closure
- Collector Activated vs Donor Activated
- Easy to Photocopy Results
- Built-In Adulteration Testing Available
- FDA Approved

RESULTS YOU CAN TRUST

POINT OF CARE MEDICAL
DIAGNOSTIC TESTING PRODUCTS

NOBLE




MEDICAL INC

SPLIT SPECIMEN CUP®



MULTI-PANEL CUP

TEST	CALIBRATOR	CUT-OFF (NG/ML)
AMP	Amphetamine	1,000
AMP 300	Amphetamine	300
BAR	Barbiturates	300
BUP	Buprenorphine	10
BZO	Benzodiazepines	300
COC	Cocaine	300
COC150	Cocaine	150
EDDP	Methadone Metabolite	300
mAMP	Methamphetamine	1,000
mAMP500	Methamphetamine	500
MDMA	Ecstasy	500
MOP	Morphine	300
MTD	Methadone	300
OPI	Opiates	2,000
OXY	Oxycodone	100
PCP	Phencyclidine	25
PPX	Propoxyphene	300
TCA	Tricyclic Antidepressants	1,000
THC	Marijuana	50

1 COLLECT SPECIMEN

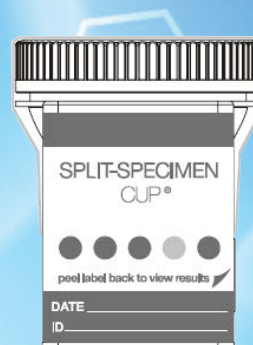


2 INSERT KEY

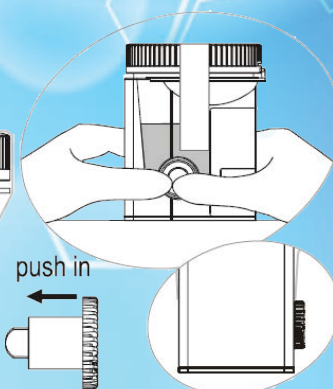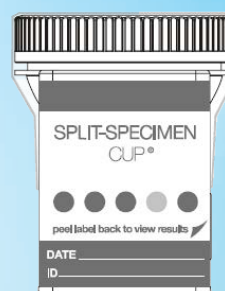

3 PEEL BACK THE LABEL



4 READ THE RESULTS

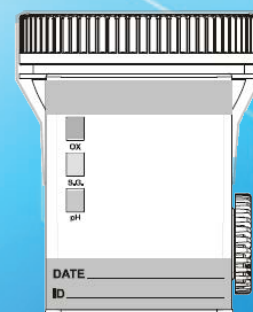

877.836.5713 NOBLEMEDICAL.COM
 19525 Janacek Court, Ste. 104 | Brookfield, WI 53045

NOBLE
 MEDICAL INC